SCALE 1:1

**TEMPLATE
WALL BRACKET
LSU**

ENGLISH

MOUNTING INSTRUCTIONS - WALL BRACKET

Please refer to the enclosed template while following the instructions below. (The template is a 1:1 scale and can be used directly on the wall when mounting the Wall Bracket).

- Choose a suitable flat location (to ensure proper charging) with a sufficiently clear area for the installation of the Wall Bracket.
- Partially install upper screws, mount the Wall Bracket and move it downward into position [1].
- Install lower screw [2], and tighten the upper screws.

Caution: Make sure that the screws provided are appropriate for the wall material

INSTALLATION/REMOVAL OF LSU IN WALL BRACKET

Installation:

Place the bottom of the LSU on the Wall Bracket Feet [3] and press the LSU firmly into place. A correct installation is confirmed by a "click", and causes the "External Power Indicator" LED on the LSU to come on.

One-hand release function:

Remove the LSU from the Wall Bracket by holding firmly on the handle while using the thumb to press down the Release Tab [4].

Caution: Removing the LSU without pressing down the Release Tab, could cause damage to the fastening brackets (Cat.no.78 04 36) on the LSU and thus lead to improper charging.

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